MEDICALPROGRAM

MEDICAL PROGRAM

 2012 

**ALPRO**[®]
ALPRO MEDICAL GMBH
Germany

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ALPRO is one of the most creative companies in the infection control sector. **ALPRO** stands out due to its first class and environment-friendly products and its reliability. **ALPRO** is also one of the world's leading specialists in the cleaning, maintenance and disinfection of dental suction units with and without amalgam separator (*AlproJet-D, AlproJet-DD, AlproJet-W, AlproCleaner*).

Aldehyde-free products are a matter of course for us like the long lasting practice-proven **ALPRO**-products *MinutenSpray-classic* and *MinutenWipes* for rapid disinfection of surfaces, the universal applicable products *BIB* and *BIB forte* (drills and instruments bath) for disinfection and cleaning of **all instruments** as well as *PlastiSept* for the gentle disinfection of all plastics and plastic surfaces.

ALPRO is a global leader in the research and development of products for removing biofilm in dental unit waterlines and sanitation of treatment water.

Drinking water quality in dental units can be achieved with *BRS* (*Biofilm-Removing-System*) and *Alpron/Bilpron*. In order to maintain this standard (which is necessary for the practices) two treatment units *Weekend II* and *BCS* (*Bottle-Care-System*) are available for connection to the dental units.

The cleaning and disinfection solutions of the *WL-range* for manual processing (internal cleaning and disinfection) of handpieces and turbines are a real highlight. These solutions demonstrably comply with the highest cleaning (*DIN ISO 15883*) and disinfection requirements and have been tested and approved by several manufacturers.

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Alprox

Universal rinsing solution ready for use, also for application in dental water jets and in ultrasonic scalers.

Alprox is used for the cleaning and care of the oral cavity as well as of gingival pockets during and after treatment of parodontosis. Alprox can also be used as rinsing solution for the cleaning and care of gingival pockets during/after removal of tartar and concretions and for rinsing the root canal. Reduces surface tension before taking impressions for more defined contours.

Application

For mouth rinsing rinse/gargle 1 minute with 20 ml undiluted. Do not swallow, do not rinse with water afterwards.

Activity basis

- modern combination of biguanid and tosykloramid

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Efficacy

- bactericidal incl. MRSA
- yeasticidal
- virus-inactivating: HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1) as well as non-enveloped Adeno viruses
- Proved according to EN 1500, EN 14476, EN 1040 and EN 1275

Activity basis

- ethanol, propan-2-ol, skin care substances, PHB esters
AlproSept-HDE gel incl. thickener

AlproSept-HDE/AlproSept-HDE gel

Hygienic and surgical hand disinfection as well as skin disinfection before carrying out ordinary injections and punctures of peripheral vessels.

AlproSept-HDE and AlproSept-HDE gel have a particularly good cutaneous tolerance.
Free of aldehydes and phenols.

Application

AlproSept-HDE/HDE gel is applied undiluted.

For hygienic hand disinfection rub 3-5 ml AlproSept-HDE/HDE gel on the hands for 30 seconds and keep it moist.

For surgical hand disinfection rub 3-5 ml **per hand** AlproSept-HDE/HDE gel up to the elbows for 5 minutes and keep moist.

For skin disinfection before carrying out ordinary injections and punctures of peripheral vessels rub with a cellulose wipe soaked with AlproSept-HDE/HDE gel on the spot you want to disinfect.

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ca. 5 sekunder

Hygienic hand disinfection

Standard rubbing method for hygienic hand disinfection according to EN 1500

For the **hygienic hand disinfection** apply the hand disinfectant into your cupped, dry hands and rub for **30 seconds** following the listed steps up to the wrists. Every move of step has to be repeated five times. After finishing step 6 repeat the single steps until the indicated rubbing time has been attained.

Pay attention to keep moist the hands over the entire rubbing time. If necessary, take more hand disinfectant.

Step 1

Rub the hand palms together additionally the wrists

ca. 5 seconds

Step 2

Rub right hand palm against the back of the left hand and vice versa

ca. 5 seconds

Step 3

Palm to palm with spread fingers

ca. 5 seconds

Step 4

Rub the outsides of the folded fingers against the opposite palm

ca. 5 seconds

Step 5

Rub the left thumb in the closed right hand fist and vice versa

ca. 5 seconds

Step 6

Move the closed fingertips in circles in the right palm and vice versa

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REF 3518 1 Ltr.*

CREMANA®-wash

Mild washing lotion for frequent hand washing. Cleans the skin gently. Skin typical pH value. Maintains the protective acid mantle of the skin. Does not form any lime scale soap residues in the washbasin or deposits in the draining system.

Field of application

Particularly mild cream soap for sensitive hygiene areas such as dental and medical practices, hospitals etc.

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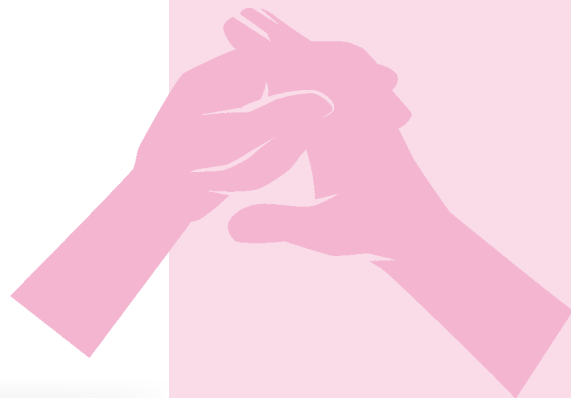
REF 3507 300 ml*

**CREMANA®-derm protect**

Greasing skin protection and barrier cream (W/O) free of silicone, for application at work in case of frequent contact with water based products, like cleaning and disinfection solutions. Ideal for skin protection when wearing gloves.

Application and dosage

Apply evenly on dry and cleaned skin. For the protection of both hands disperse an approximately peanut-sized quantity from the backs of the hands to the palms.

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REF 3508 35 ml*

CREMANA®-Creme-Lotion

Cream for intensive skin protection and care. The cream lotion is easy to rub, absorbs quickly and does not grease. Suitable for every type of skin.

Application and dosage

After the cleaning/decontamination of the hands (breaks/end of work) put CREMANA Creme-Lotion (quantity of cherry pits) on the back of the hand and rub thoroughly.

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- REF 3006 500 ml*
- REF 3005 5 Ltr. Canister*
- REF 3007 5 Ltr. Cubitainer*

Efficacy/Certification

- bactericidal incl. TBC and MRSA
- yeasticidal
- virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia as well as non-enveloped Adeno viruses
- with remanence effect (reduction of recontamination)
- certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for surface disinfection in the medical area

Activity basis

- modern combination of alcohols, QACs and alcyamines

MinutenSpray-classic

Aldehyde-free and phenol-free quick-acting disinfectant ready for use for surfaces of medical devices e. g. turbines, contra-angles and hand pieces, unit grips and suspension tables; contains active cleaning ingredients. Approved and recommended amongst others by Sirona Dental.

Application

Completely moisten surfaces and objects that have to be disinfected and let the disinfectant take effect according to the spectrum of activity. Afterwards wipe off with a disposable wipe (e. g. MinutenWipes). Only for use on alcohol compatible surfaces.

Contact times (undiluted)

- bactericidal (incl. TBC, MRSA) 1 minute
- yeasticidal 1 minute
- virus-inactivating: 30 seconds
enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia
non-enveloped Adeno viruses 2 minutes

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Efficacy/Certification

- bactericidal incl. TBC and MRSA
- yeasticidal
- virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia as well as non-enveloped Adeno viruses
- with remanence effect (reduction of recontamination)
- solution certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology) meets the DGHM requirements and EN-standards for surface disinfection in the medical area

Activity basis

- modern combination of alcohols, QACs and alclamines

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BigBox 20 x 120 CidWipes Refill*

Efficacy/Certification

- bactericidal incl. TBC and MRSA
- yeasticidal
- virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia as well as non-enveloped Adeno viruses
- with remanence effect (reduction of recontamination)
- solution certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology) meets the DGHM requirements and EN-standards for surface disinfection in the medical area

Activity basis

- modern combination of alcohols, QACs and alcyamines

CidWipes eco

Aldehyde-free wet wipes for a quick-acting disinfection of surfaces of medical devices such as turbines and contra-angles, unit grips and suspension tables; contains active cleaning ingredients. Approved and recommended amongst others by Sirona Dental.

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REF 4650 500 ml*
REF 3063 5 Ltr. Canister*
REF 3062 5 Ltr. Cubitainer*

PlastiSept

Alcohol-free and aldehyde-free disinfectant ready for use. Especially suitable for alcohol sensitive surfaces of medical devices such as treatment units (e. g. seat cushions, head rests), suspension tables, in the X-ray area (e. g. chin rests at OPG), also ideal for acrylic glass (also solarium beds). Approved and recommended amongst others by Sirona Dental.

Application

Completely moisten surfaces and objects to be disinfected and let the disinfectant take effect according to the spectrum of activity.

Afterwards wipe off with a disposable wipe (e. g. PlastiSept-Wipes).

Contact times (undiluted)

- bactericidal incl. MRSA 1 minute
- yeasticidal 1 minute
- virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia 1 minute

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Efficacy/Certification

- bactericidal incl. MRSA
- yeasticidal
- virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia as well as non-enveloped Noro viruses
- with remanence effect (reduction of recontamination)
- certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology) meets the DGHM requirements and EN-standards for surface disinfection in the medical area

Activity basis

- modern combination of quaternary ammonium compounds (QAC)

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Efficacy/Certification

- bactericidal incl. MRSA
- yeasticidal
- virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia as well as non-enveloped Noro viruses
- with remanence effect (reduction of recontamination)
- solution certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology) meets the DGHM requirements and EN-standards for surface disinfection in the medical area

Activity basis

- modern combination of quaternary ammonium compounds (QAC)

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PlastiSept-Wipes

Alcohol-free and aldehyde-free wet wipes especially for alcohol sensitive surfaces of medical devices such as treatment units (e. g. seat cushions, head rests), suspension tables, in the X-ray area (e. g. chin rests at OPG), also ideal for acrylic glass (suited for solarium). Approved and recommended amongst others by Sirona Dental.

Contact times (undiluted)

- bactericidal incl. MRSA 1 minute
- yeasticidal 1 minute
- virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia 1 minute

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MaxiWipes-L

160 MaxiWipes-L in Box, torra/dry*
4 x 160 MaxiWipes-L Refill torra/dry*
(50 g/m², 20 x 40 cm) for high
absorption and maximum application
of the active solution

MaxiWipes/MaxiWipes-L 

Large dry wipes in the MaxiWipes-Box for individual preparation of wet wipes for using MinutenSpray-classic or PlastiSept; for the cleaning and disinfection of surfaces of medical devices.

Field of application, soaked with PlastiSept

MaxiWipes soaked with PlastiSept are especially suited for particularly sensitive surfaces which are sensitive to alcohol such as treatment units (e. g. seat cushions, head rests), suspension tables, in the X-ray area (e. g. chin rests at OPG), also ideal for acrylic glass (suited for solarium).

Contact times with PlastiSept

- bactericidal incl. MRSA 1 minute
- yeasticidal 1 minute
- virus-inactivating:
enveloped viruses incl. HBV, HCV, HIV,
Herpes simplex, Influenza (incl. H1N1, H5N1),
BVDV, Vaccinia 1 minute

Field of application, soaked with MinutenSpray-classic

MaxiWipes soaked with MinutenSpray-classic are suited for surfaces which are compatible with alcohol if quick drying and a comprehensive disinfection is wanted.

Contact times with MinutenSpray-classic

- bactericidal (incl. TBC and MRSA) 1 minute
- yeasticidal 1 minute
- virus-inactivating: 30 seconds
enveloped viruses incl. HBV, HCV, HIV,
Herpes simplex, Influenza (incl. H1N1, H5N1),
BVDV, Vaccinia
non-enveloped Adeno viruses 2 minutes

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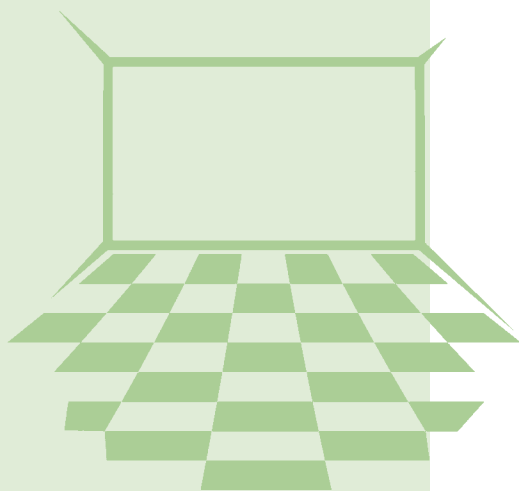
REF 3027 1 Ltr.*
REF 3028 5 Ltr.*

Efficacy

- bactericidal incl. MRSA
- fungicidal (*A. niger*, *C. albicans*)
- virucidal according to the requirements of RKI (enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (inkl. H1N1, H5N1), BVDV, Vaccinia as well as non-enveloped viruses as Adeno, Noro, Polio and Rota)
- sporizid against clostr. diff. (EN 13704)

Activity basis

- modern combination of QAC

DesNet +

Liquid concentrate free of aldehydes and phenols for the reliable cleaning and disinfection of surfaces of medical devices in the medical and dental practice with an extensive microbicidal effectiveness.

Concentrations

- General hygiene + prophylaxis (incl. enveloped viruses):
0,5 % - 60 min. / 2,0 % - 15 min.
- Critical sectors (e. g. operation room) for a comprehensive virucidal action (enveloped and non-enveloped viruses):
4,0 % - 60 min. / 5,0 % - 15 min.

Environment compatibility

DesNet + solutions are ecologically harmless as all active agents have been classified as easily biodegradable according to modified OECD-screening-test.

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REF 4151 Box 200 CleanWipes*
REF 4152 Carton 12 x 200 Wipes Refill*

CleanWipes

Alcohol- and aldehyde-free wet wipes especially for the non protein fixing cleaning previous to disinfection of surfaces of medical devices such as transmission instruments, general instruments, endoscopes, also for the cleaning of plastic surfaces of treatment units.

Field of application

CleanWipes are ready for use wet wipes with maximum cleaning efficacy und material compatibility. Especially developed for the removal of protein residues from surfaces of general medical instruments (forceps, probes, tweezers, transmission instruments, endoscopes) and surfaces of treatment units.

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Efficacy

- cleaning without fixing protein
- bacteriostatic
- fungistatic

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Efficacy

- removes powerful and gently proteins, residues of tissues, biofilm etc. from surfaces of medical devices

Activity basis

- phosphates, tensides, parabenes, defoaming agents, stabilizers, coated enzymes

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AlproZyme

Highly effective alkaline-enzymatic granulate for the cleaning of medical and dental instruments, rotating precision instruments and endoscopes.

Application

Cleaning granulate for the intensive pre-cleaning in soaking bath, ultrasonic systems, washers and washer disinfectors for the removal of proteins, residues of tissues, biofilm etc. from:

1. General dental and medical instrument (diagnostic, preserving and surgical instruments e.g. extracting forceps, root elevators, dental mirrors, rigid and flexible endoscopes etc.)
2. Dental rotary precision instruments e. g. steel, hard and diamond drills, surgery drills and carbide burs, crown cutters, root canal instruments, silicone polishers etc.

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500 ml*

IC-100 Instrument Cleaner

Cleaning foam ready for use for the non protein fixing pre-cleaning of instruments. Dissolves residues of protein and other body fluids from surfaces of medical products.

Field of application

Non protein fixing pre-cleaning of

1. general dental and medical instruments (diagnostic and conserving instruments such as forceps, root elevators, mouth mirrors, rigid and flexible endoscopes etc.)
2. rotating dental instruments such as steel, hard metal and diamond drills and burs, milling cutters, instruments for root canals and crowns, rubber burnishers etc.

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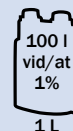
REF 4018 4 Ltr. Balja/Tray*

Efficacy/Certification

- bactericidal incl. TBC and MRSA
- yeasticidal
- Virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia as well as non-enveloped Polio viruses (55°C)
- Meets the CJD-prophylaxis according to the recommendation of RKI (pH>=10)
- complies with the requirements of the VAH (Association for Applied Hygiene)/ DGHM (German Society for Hygiene and Microbiology) for instrument disinfection at **high** organic load

Activity basis

- modern combination of QAC and alcyamines

CE 0123

REF 3024 1 Ltr.*
REF 3025 5 Ltr.*

BIB forte

High performance disinfection

Aldehyde- and phenol-free liquid concentrate for the non protein fixing cleaning and disinfection of medical and dental instruments as well as rotating precision instruments. Also given material compatibility with endoscopes.

Approved and recommended by: Acurata, Busch & Co., Dentsply Maillefer, Elma, Fujinon Corporation/Japan, Gebr. Brasseler (Komet), Henke-Sass, Wolf, Karl Storz, SS White and VDW.

Field of application

Highly efficient liquid concentrate for the cleaning and disinfection of

1. dental rotary precision instruments as e. g. steel, hard, diamonds, surgery drills and carbide burs, crown cutters, root canal instruments etc.)
2. general dental and medical instruments (diagnostic, preserving and surgical instruments e. g. extracting forceps, root elevators, dental mirrors, rigid and flexible endoscopes, etc.)

Application concentrations

- bactericidal (incl. TBC, MRSA), yeasticidal:

efficient	1%	60 minutes
quick	2%	5 minutes in ultrasonic devices
	3%	30 minutes
	4%	15 minutes
- virus-inactivating:

enveloped viruses incl. HBV/HCV/HIV, Herpes simplex, Influenza (H1N1, H5N1), BVDV, Vaccinia non-enveloped	1%	5 minutes
Polio viruses:	4%	5 minutes at 55°C (e.g. in ultrasonic device)

Usage durability

1 week durability in use in case of normal contamination

Compatibility

BIB forte is a carefully adjusted formula with extraordinary corrosion inhibiting qualities, thus no rust problems will occur on steel drills and instruments which are sensitive to corrosion.

Environment compatibility

BIB forte is completely free of aldehydes and phenols, stands for high effectiveness, safe application and efficiency (application in a solution ready for use at 1 – 4 %). It is very compatible with materials as it does not contain any alkalis, chlorinated hydrocarbons and halogens.

BIB forte is biodegradable according to OECD.

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WL-clean

All-purpose rinsing and cleaning solution ready for use, free of aldehydes and alcohol, for the **non protein fixing** inner cleaning and decontamination of hollow part instruments, e. g. turbines, hand pieces, contra-angles, internally cooled instruments and endoscopes prior to disinfection/sterilisation.

WL-clean is a Medical Device of class I. The proof of suitability for cleaning has been given according to the series of standards DIN ISO 15883 and according to the specifications of the German Society for Sterile Goods Supply (DGSV) and has been confirmed by an expert's report.

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- Biguanider, PHB-ester

Efficacy

- bacteriostatic
- fungistatic
- tested under practice conditions with soiling test according to DIN ISO/TS 15883-5

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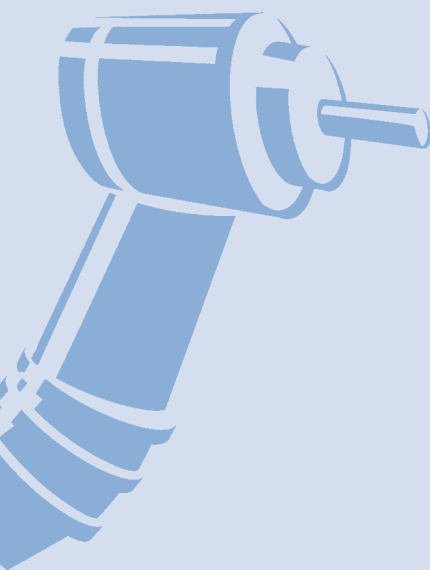
Efficacy/Certification

- bactericidal incl. TBC and MRSA
- yeasticidal
- virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1). Testing with BVDV, Vaccinia, Adeno as well in practice test (phase 3) in combination with WL-clean with MS2-phages (surrogate for noro- and polio viruses)
- tested according to surface and instrument disinfection criteria and under practical conditions with soiling test according to DIN ISO/TS 15883-5.
- Solution certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), fulfils the DGHM requirements and EN-standards

Activity basis

- modern combination of alcohols, QACs and alcyamines

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WL-cid

All-purpose disinfecting solution ready for use, free of aldehydes, for inner disinfection of dental hollow part instruments, e. g. turbines, hand pieces, contra-angles, after the non protein fixing pre-cleaning with WL-clean.

WL-cid is a Medical Device of class II b. The disinfection capacity under simulated practical conditions according to DIN ISO 15883 has been confirmed by an expert's report. The disinfection capacity according to the catalogue of requirements of the DGHM for instrument disinfection has been confirmed by an expert's report.

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REF 3170 4 x 300 ml*

WL-dry 

Medical propellant for final blow and quick drying of hollow part instruments, e. g. turbines, hand pieces and contra-angles, after use of WL-clean/WL-cid**.

Field of application

Removing of residues by cleaning and disinfection agents and care products.

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REF 4160 WL-Blow Set*

WL-Blow 

The compressed air gun WL-Blow with integrated air flow limiter is used to blow out, to dry and to support the cleaning of hollow part instruments and transmission instruments after cleaning and disinfection with WL-clean/WL-cid**. WL-Blow is intended for use in the processing room and can be connected to the existing oil-free dry air compressor with bacterial filter!

Field of application

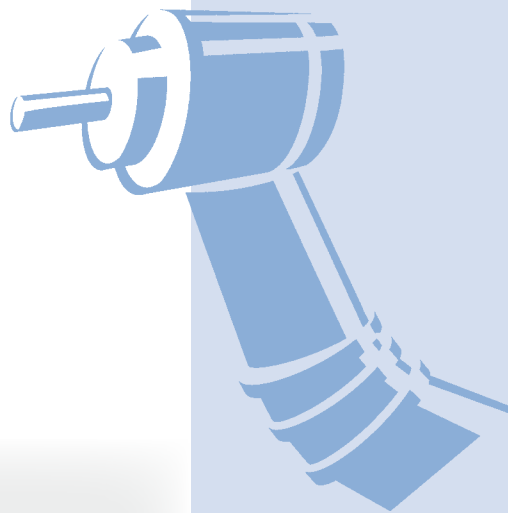
Removing of residues by cleaning and disinfection agents and care products in hollow part instruments and transmission instruments.

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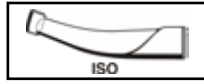
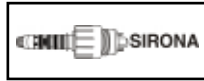
REF 3530

WL-adapters

Universal adapters for the WL-series:

WL-Adapter 01** (plastics)

Universal adapter with Luer connector for transmission instruments (ISO 9168) for blowing-out of cleaning and maintenance products (WL-clean, WL-cid, maintenance oil) with WL-dry/WL-Blow. Also suitable for blowing out using the multifunction syringe and to bring in the cleaning solution WL-clean via Luer connection e.g. into general hollow part instruments.

Adapters for spray water and spray air channels:

WL-Adapter 02** (stainless steel)

Adapter for spray water and spray air channels of all **hand pieces and contra-angles with ISO-interface**.

REF 3522

WL-Adapter 03** (stainless steel)

Adapter for spray water and spray air channels of **turbines with "Sirona" interface**.

REF 3523

WL-Adapter 04** (stainless steel)

Adapter for spray water and spray air channels for **turbines with "Multiflex"- (KaVo/SciCan) interface**.

REF 3524

WL-Adapter 06** (stainless steel)

Adapter for spray water and spray air channels for **turbines with "Roto-Quick"-interface (W&H)**.

REF 3526

WL-Adapter 07** (plastics)

Adapter for long shank contra-angles solely suitable for spray water and spray air channels of **Sirona T1 Classic-Line**.

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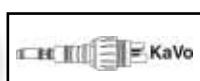
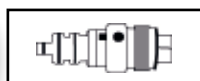
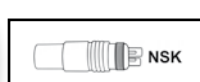
REF 3541

WL-adapters

Adapters for complete internal cleaning:

WL-Adapter 02/G** (stainless steel)

Adapter for **hand pieces and contra-angles with ISO-interface** like adapter 02 but for complete internal cleaning/disinfection incl. head of instrument.

WL-Adapter 03/G** (stainless steel)

Adapter for **turbines with „Sirona“ interface** like adapter 03 but for complete internal cleaning/disinfection incl. head of instrument.

WL-Adapter 04/G** (stainless steel)

Adapter for **turbines with „Multiflex“- (KaVo/SciCan) interface** like adapter 04 but for complete internal cleaning/disinfection incl. head of turbine.

WL-Adapter 05/G** (stainless steel)

Adapter for **turbines with „Bien Air“-interface** for complete internal cleaning/disinfection incl. head of turbine.

WL-Adapter 06/G** (stainless steel)

Adapter for **turbines with „Roto-Quick“-interface (W&H)** like adapter 06 but for complete internal cleaning with WL-clean/WL-dry/WL-Blow incl. head of turbine.

WL-Adapter 09/G** (stainless steel)

Adapter for **turbines with NSK-interface** for complete internal cleaning/disinfection incl. head of turbine.

WL-Adapter 10/G** (stainless steel)

Adapter for **turbines with Morita-interface** for complete internal cleaning/disinfection incl. head of turbine.

WL-Adapter 11/G** (stainless steel)

Adapter for **ACTEON-AIR N GO Airpolisher „Supra/Perio nozzle“** for complete internal cleaning/disinfection with WL-clean, WL-cid, WL-dry

Introductory set with 2 adapters (adapter 02/G, 04/G) incl. 1 x replacement set of o-rings

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The application of the WL-series

Option 1: Manual cleaning with steam disinfection* (unpacked)

- Step 1** Outside cleaning
- Step 2** Internal cleaning
- Step 3** Blow/dry out
- Step 4** Outside disinfection
- Step 5** Maintenance
- Step 6** Steam disinfection

Option 2: Manual cleaning and disinfection*

- Step 1** Outside cleaning
- Step 2** Internal cleaning
- Step 3** Outside disinfection
- Step 4** Internal disinfection
- Step 5** Blow/dry out
- Step 6** Maintenance

Remark:

After the inner cleaning and disinfection the instruments have to be maintained/lubricated (according to instructions of manufacturer) and sterilized if necessary, packed or unpacked, with appropriate steam steriliser (consider process instruction).

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Foto: Fa. Melag

TR-3

**Liquid cleaner for all thermal washers and washer disinfectors**

Alkaline cleaning concentrate with slightly increased viscosity, therefore universally applicable in automatic and semi-automatic dosage systems for mechanical processing of general and rotary instruments in clinics and practices.

For washers and washer disinfectors.

Field of application

- alkaline cleaner for very good removal of blood and protein residues at all temperature programs
- especially gentle to materials
- protects instruments made of steel and stainless steel from corrosion
- suitable in combination with TR-Neutralizer also in washer disinfectors as e. g. Miele Thermodisinfectors

Dosage

Dosage: 0,3 % - 0,5%

Fill the concentrate into the door dosage system of the cleaner and close the flap.

For automatic dosage systems connect the dosage lance with machine-compatible can and if necessary adjust the dosage quantity.

Use TR-Clear for the clear rinsing.

Activity basis

- *tensides, complexing agents, phosphates*

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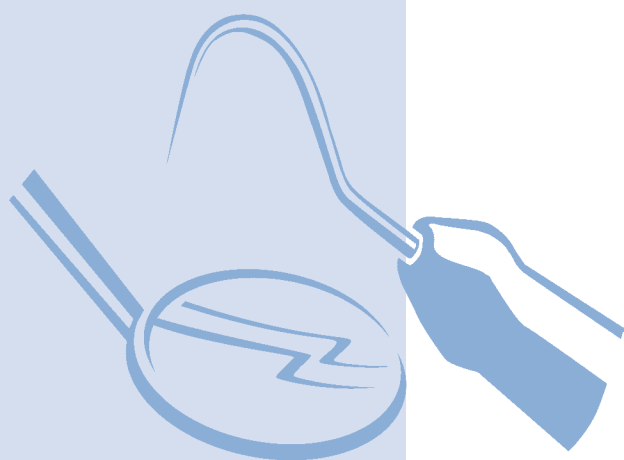
- phosphoric acid

TR-Neutralizer

Liquid concentrate for neutralization

Liquid concentrate for neutralization after the alkaline cleaning with TR-3 for the mechanical preparation of general and rotary instruments in clinics and practices. Approved for Miele Professional G 7735, G 7781, G 7881, SIROCLEAN.

Field of application

Neutralization component based on phosphoric acid for use in cleaning and disinfecting machines during thermal preparation in the Vario TD program or RKI procedure for neutralization after the alkaline cleaning.

Also suitable for basic cleaning of dental instruments as immersion bath (see instructions of the manufacturer).

Dosage

1 - 3 ml/l, automatically via the dosage pump of the thermal disinfecter. For basic cleaning in the immersion bath prepare cold water, add 1 - 3 % TR-Neutralizer and afterwards put the instruments into the bath. Contact time max. 1 hour. Afterwards rinse thoroughly with water.

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REF 3720 1 Ltr.*

TR-Clear

Liquid rinsing and neutralizing concentrate for washers and washer disinfectors

Liquid rinsing concentrate for the mechanical processing of general and rotary instruments in clinics and practices.

Approved for Miele Professional G 7735, G 7781, G 7881, SIROCLEAN.

Field of application

For rinsing during mechanical preparation of instruments, trays, rotating instruments etc. in clinics and practices.

Dosage

In cleaning machines/thermal disinfectors:
2 - 6 ml/l automatically via the door dosage unit.

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- Tensider, fosfonsyra

Efficacy

- reduces surface tension of the rinsing water (no spots)
- neutralizes after the alkaline cleaning

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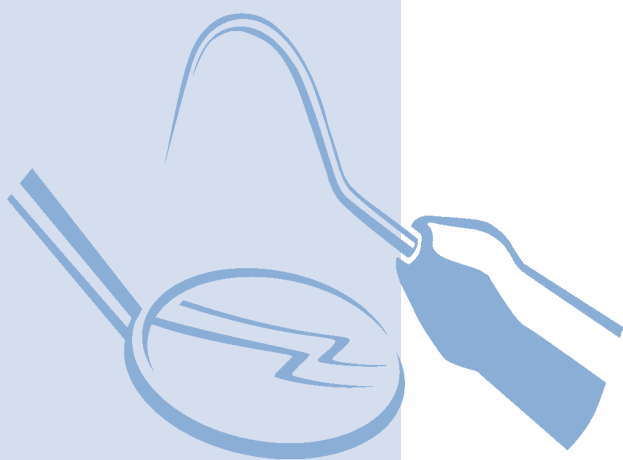
REF 3730 5 kg*

TR-Salt

Regeneration salt for water softener in washers and washer disinfectors

Coarse-grained special salt for water softener to prevent calcium deposits in medical devices and on the items.

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AlproJet-D

Highly efficient aldehyde-free and non-foaming liquid concentrate for the daily cleaning, disinfection and maintenance of dental aspiration and separation systems. Approved and recommended amongst others by Sirona Dental.

Dosage**

Prepare a 5 % solution (50 ml of AlproJet-D concentrate and 950 ml of water result in 1 litre of solution ready for use) and rinse 1 - 2 x a day (at noon/evening each time with 1 ltr.). Rinse with the **AlproJet MixCup** (capacity 2 Ltr.) through the **aspirating tubes** (1 Ltr.) and **spittoon bowl drain** (1 Ltr.). Afterwards let the solution take effect and rinse well with water only before starting to work. Dental units with chemical cleaning of aspiration tubes: Please follow the manufacturer's instructions.

Compatibility

AlproJet-D is highly compatible with materials such as plastics, rubber and non-ferrous metals, alloyed steel etc.

Environment compatibility

AlproJet-D does not contain any chlorine or oxygen splitters and is free of aldehydes and phenols. Biodegradable (according to OECD regulations).

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Efficacy

- bactericidal
- yeasticidal
- virus-inactivating: HCV (BVDV)

Activity basis

- modern combination of QAC and tensides

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REF 3085 AlproJet MixCup*

Efficacy/Certification

- bactericidal
- yeasticidal
- Virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia as well as non-enveloped Adeno viruses
- certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for the disinfection of instruments

Activity basis

- modern combination of QAC and tensides

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AlproJet-DD

Highly concentrated liquid free of aldehydes and phenols for daily cleaning, maintenance and disinfection of all dental aspiration and separation systems. Approved and recommended amongst others by Sirona Dental.

Dosage**

Prepare a 2 % solution (20 ml of AlproJet-DD concentrate and 980 ml of water result in 1 litre of solution ready for use) and rinse 1 - 2 x a day (at noon/evening each time with 1 ltr.). Rinse with the **AlproJet MixCup** (capacity 2 Ltr.) through the **aspirating tubes** (1 Ltr.) and **spittoon bowl drain** (1 Ltr.). Afterwards let the solution take effect and rinse well with water only before starting to work.
Dental units with chemical cleaning of aspiration tubes: Please follow the manufacturer's instructions.

Contact times

- | | | |
|---|-----|------------|
| • bactericidal/yeasticidal | 2 % | 60 minutes |
| • virus-inactivating: enveloped viruses | 2% | 10 minutes |
| non-enveloped Adeno viruses | 2% | 15 minutes |

Compatibility

The active substances, harmonizing perfectly together, are particularly compatible with materials.

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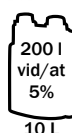
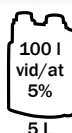
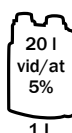
AlproJet-W

Highly efficient aldehyde-free and non-foaming liquid concentrate for the weekly contra-cleaning and maintenance of dental aspiration and separation systems. Approved and recommended amongst others by Sirona Dental.

Dosage**

To be applied twice a week. In case of high protein load AlproJet-W can also be used every day.

Prepare a 5 % solution (50 ml AlproJet-W concentrate and 950 ml warm water, result is 1 litre solution ready for use each time with 1 ltr.). Rinse with the **AlproJet MixCup** (capacity 2 Ltr.) through the **aspirating tubes** (1 Ltr.) and **spittoon bowl drain** (1 Ltr.). Afterwards let the solution take effect and rinse well with water before starting to work.

Compatibility

Very good compatibility with materials

Environment compatibility

AlproJet-W does not contain any chlorine or oxygen splitters and is free of aldehydes and phenols. Biodegradable according to OECD regulations.

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Efficacy

- cleaning and disinfecting (in combination with AlproJet-D)

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- bactericidal
- yeasticidal

Activity basis

- modern combination of alcohol and QAC

AlproCleaner

Aldehyde-free alkaline liquid cleaner with disinfecting effect ready for use for the cleaning of spittoon bowls and spittoon bowl drains of all dental treatment units supplied with and without amalgam separators.

Approved and recommended amongst others by Sirona Dental.

Dosage**

Wipe the spittoon bowl with 10 - 20 ml of (undiluted) AlproCleaner. Rinse with clear water.

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REF 4500 5 Ltr.*
REF 4017 1 Ltr. Wanne/Tray*

PrintoSept-ID

Disinfection and cleaning solution ready for use, free of aldehydes and phenols, for all kinds of impressions, worn dental prostheses as well as dental workpieces by immersion.

Field of application

Disinfecting solution ready for use as immersion bath with certified dimension stability for all dental impressions made of alginate, silicone, polyether, hydrocolloid, etc. Also suitable for worn dental prosthesis as well as dental workpieces. PrintoSept-ID can also be used in disinfection devices for the processing of impressions, models, bite templates, auxiliaries, etc.

Contact times

- bactericidal 1 minute
- yeasticidal 1 minute
- tuberculocidal 5 minutes
- virus-inactivating: enveloped viruses 5 minutes

Usage durability

1 week durability in use by normal contamination (scientific approval)

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Efficacy/Certification

- bactericidal (incl. TBC)
- yeasticidal
- Virus-inactivating: enveloped viruses incl. HBV, HCV, HIV, Herpes simplex, Influenza (incl. H1N1, H5N1), BVDV, Vaccinia
- certified by VAH (Association for Applied Hygiene)/DGHM (German Society for Hygiene and Microbiology), meets the DGHM requirements and EN-standards for the disinfection of instruments

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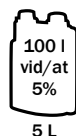
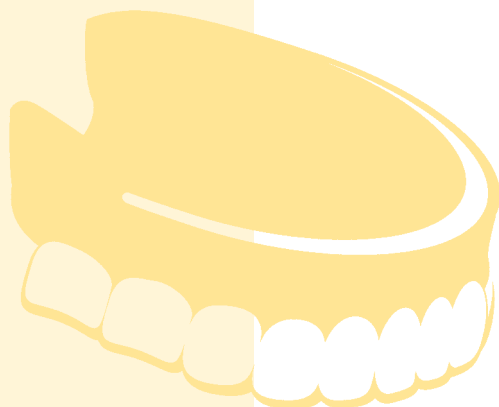
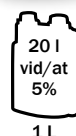
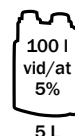
1 Ltr.*
5 Ltr. dunkar*
5 Ltr. Cubitainer*

Activity basis

- Sodium alkanesulfonate, complexing agents

AlproSol

Liquid concentrate of active ingredients for the cleaning of impression trays, mixing-spatulas and instruments.

Field of application

AlproSol dissolves zinc oxide eugenol, carboxylate-, phosphate- and glass-ionomer cements as well as adhesive varnishes and alginates that are left as residues on instruments, mixing-spatulas and impression trays.

Dosage

Dilute AlproSol with water to a 3-5% application solution (30-50 ml per litre of solution). Clean slightly impression trays and instruments and put them over night into the solution.

Ultrasonic cleaning

AlproSol can also be used in ultrasonic devices.

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AlproSolvent

Cleaning solution ready for use, based on natural orange-oils. Solves pastes and impression materials (alginate, silicone, polyether or polysulfide) as well as from adhesive varnishes and glue residues from impression trays and instruments.

Also suitable for removing cements, zinc-oxide-eugenol pastes and plaster.

Application AlproSolvent

Put approx. 5 - 10 ml of the cleaning solution on a wipe and rub the surfaces and objects to be cleaned.

Activity basis

- orangeterpenes in organic solvents

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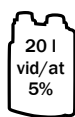
- inorganic acids and tensides

TarClean

Acid liquid concentrate for the removal of tartar/cements from dental-prostheses and dental instruments. Removes temporary dental fillings, metal oxides and investment materials.

Dosage

Dilution with water at 10 - 20 %. Application of 5 % in ultrasonic devices and in case of minor soiling. Afterwards rinse the cleaned object thoroughly with water.

Compatibility

Suitable for stainless steel, precious metal, plastics and intact aluminium parts

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SteriCleaner

Solution ready for use for the cleaning of the sterilizing chamber, instrument trays and steel surfaces.

Field of application

- Removes residues of blood, disinfectants (instrument disinfection) and tissue residues (of nerve broaches etc.)
- Protects unpacked instruments from soiling and unhygienic deposits
- In combination with **Sterisponge (REF 3129)** powerfully removes all residues which have formed during the sterilization process, even if they are old or heavily burned into material
- Cleans powerfully with best material compatibility, free from grinding parts

Activity basis

- modern combination of tensides and alcohol

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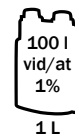
REF 3184 4 x 1 Ltr.*
REF 3177 Alpron StarterSet*

Alpron

Liquid concentrate for decontamination and disinfection for optimal hygiene of process water in dental unit water lines with and without dosing device. Approved and recommended by notable dental unit manufacturers.

Field of application

Alpron is a concentrated liquid for the continuous decontamination of process water in DUWL of dental treatment units (including the instrumentation and the tumbler filler).

- Prevents the regrowth of germs
- Removes and prevents the formation of algae
- Prevents the sedimentation of lime and deposits
- Non-corrosive

Concentration (Application with BCS)

1 % for decontamination of treatment water or as concentrate for the periodical disinfection/sanitation

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Bilpron

For the decontamination, cleaning and long-lasting prevention of biofilm formation in process water lines in medical treatment units and dental units. Approval for idle time/long term disinfection amongst others by Sirona Dental.

Field of application

Disinfection solution ready for use for the application e. g. in the Weekend-System II/BCS once to twice a week. Bilpron ensure for the disinfection and long-lasting removal and prevent the formation of biofilm in the water lines of all dental treatment units (including instruments and tumbler filler).

- Removes organic as well as inorganic residues (e.g. lime stone) in a safe and reliable way
- Effective against a broad spectrum of germs
- Prevents the new formation of biofilm
- Non-corrosive
- Ready for use
- Contact time in the dental unit: until the next working day at least 12 hours, long term conservation up to 3 months possible

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Foto: Sirona PerioScan

Activity basis

- sulfamic acid, tensides, auxiliary materials

AlproCalex

Solution ready for use for the decalcification of process water lines e. g. in Perio-Scan of Sirona and other dental cleaning systems.

AlproCalex solution is based on sulfamic acid.

Field of application

Dissolves safely and gently limescale and mineral deposits from process water lines of medical equipment (e. g. ultrasonic scaler, air abrasion devices or other water-unit systems)

AlproCalex is ready for use and can be used immediately.

Higher effectiveness by increased temperature.

Compatibility

Very good material compatibility such as silicone, durethan, delrin, acetal resin, HD-PE, polyethylene HD, hostalen CYP, hostalen PPN, polypropylene, PVC, Viton, polyurethane, Cyclac, melamine resin, Terluran, Vestamid, polycarbonate, Simrit, PUR-foam, synthetic leather, acrylic glass, Lexan.

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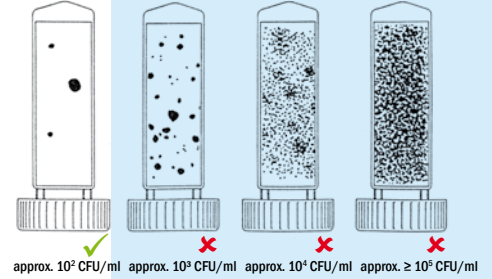
The microbial count on surfaces is measured in "colony-forming units" (CFU) per cm², in liquids in CFU per ml.

Biotest plus

Dip slide for quantitative monitoring of microbial load of water and surfaces.

Field of application

The dip slide is used for the quantitative monitoring of the microbial load for liquid and surfaces in industry, environment and medicine.

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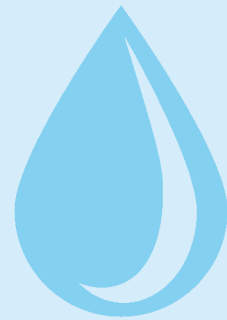
REF 3199 12 x 100 ml*

BC-San 100

Disinfecting concentrate (according to the drinking water regulation) for the cyclic decontamination/disinfection of pressure cylinders of independent water supply systems connected to medical devices (e. g. BCS).

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Bottle-Care-System with water decontamination option

Treatment water system providing an independent water supply to medical and dental treatment units according to **EN 1717/DIN 1988-4**. Opportunity of intensive decontamination and removal of biofilm from dental unit water lines.

Field of application

- Fulfills the standards DIN 1988-4 and EN 1717 (air gap)
- Each dental unit can be individually upgraded
- for the treatment processed tap water (with Alpron) or drinking water (can be used)
- The disinfection of the water lines with Bilpron is carried out weekly or during vacations

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REF 5510*

Weekend-System II

Process water line disinfection system

Chair-side device for the connection to dental treatment units. This device allows a most comfortable flooding of the process water lines with Bilpron over the weekend.

Field of application

- Easy handling
- High degree of safety and longevity
- Can be connected to each dental unit without an own decontamination system
- The disinfection of the water lines with Bilpron is carried out weekly or during vacations

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REF 4602

AlproJet MixCup

Multifunction container for the cleaning and maintenance of the aspiration system

Can tap

Dispenser tap for 3, 5 and 10 ltr. can

Cubitainer tap

Dispenser tap for 5 and 10 Ltr. cubitainer

Can wrench

Helps to unscrew the cap of a can for 3, 5 and 10 ltr.

Cubitainer wrench

Helps to unscrew the cap of a cubitainer

Impression disinfection tray

plastic, vol. 1 ltr.

For the disinfection of impressions with PrintoSept-ID thermodesinfection up to 60 °C

Disinfection tray for instruments

plastic, vol. 4 ltr.

thermodesinfection up to 60 °C

Dosage pump

For 5 and 10 ltr. cans

Dosage pump

For 1 ltr. bottle

Euro dispenser (1 Ltr.)

Multipurpose dispenser for soaps and alcoholic solutions

Measuring cup (500 ml)

For Bilpron/Weekend-System II and dosage of other disinfection solutions

Spray bottle (not shown)

500 ml empty spray bottle with spray head (for MinutenSpray-classic)

Spray head (not shown)

For 500 ml spray bottle (REF 3003)

SteriSponge

Special sponge for cleaning autoclaves with SteriCleaner

Wall holder

Wall holder for MaxiWipes-Box, MaxiWipes-L-Box

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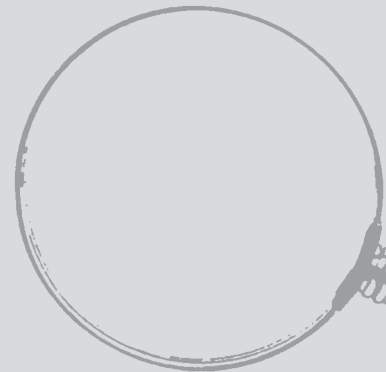
MinutenSpray-classic + MinutenWipes

- Dr. H. Brill, Hamburg, MRSA, 2009-07; EN 14561 + EN 14562, 2009-06; Langzeit/Remanenz inkl. EN 13697, 2008-01
- Prof. Dr. P. Heeg, Tübingen, DGHM, 1999-03
- Dr. J. Höffler, Hamburg, EN 1040, 2009-05; EN 1275 + EN 1276, 2009-04
- Prof. DDr. E. Marth, Graz, DGHM, 2002-07/1998-11
- Prof. Dr. R. Schubert, Frankfurt/M., VAH inkl. EN 13624 + EN 13727, 2006-05; DGHM, 1998-10
- Prof. Dr. H.-P. Werner, Schwerin, VAH, 2009-09
- Prof. Dr. G. Frösner, München, HBsAg-Verfahren, 1998-12
- Dr. J. Steinmann, Bremen, Adeno, 2006-05; Vaccinia, 2006-05; BVDV, 2003-11
Virusinaktivierend: behüllte Viren inkl. HBV, HCV, HIV, Influenza (inkl. H1N1 + H5N1), Herpes simplex, 2009-08

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ALPRO takes over responsibility for English text, not for Swedish translation.

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